



Single-use Face Mask

SOFT & EASY TO BREATHE

Non-sterile



*Picture is for reference only,
actual product may have differences.

BYD CARE 




Application

Worn by personnel to cover the mouth, nose and chin, the mask provides a physical barrier to block exhalation or ejection of pollutants from mouth and nose, and to reduce breathing in airborne particles.



Product Introduction

BYD Single-use Face Mask consists of three layers of nonwoven material:

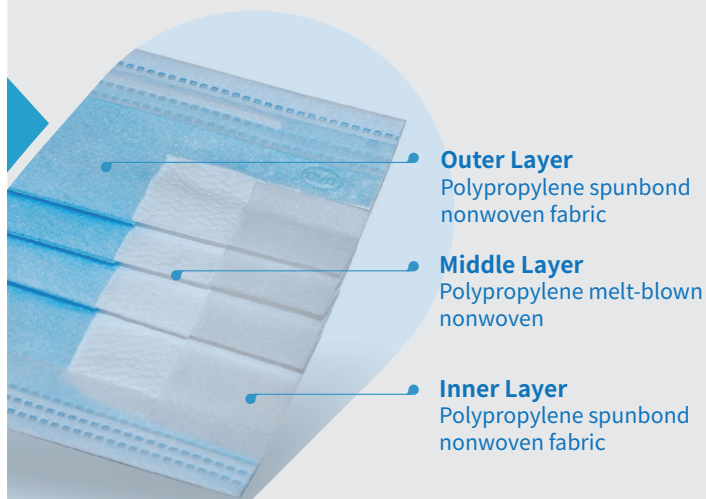

MATERIALS

Inner layer	Polypropylene spunbond nonwoven fabric
Middle layer	Polypropylene melt-blown nonwoven
Outer layer	Polypropylene spunbond nonwoven fabric
Ear loops	Polyester/nylon spandex blend
Nose clip	Metal wire with plastic covering

This product contains no components made from natural rubber latex.
Non-sterile product. One-time use only.

Wearing Instructions



1. Hold the mask by the ear loops with the colored side facing outwards and the nose bridge upwards, and place both ear loops around both ears.



2. Bend the nose clip to match the shape of the nose to prevent unfiltered air from entering.



3. Pull the mask to your chin to produce a tight seal.

Single-use Face Mask


SOFT & EASY TO BREATHE

Non-sterile



Product Specification

- 1 Mask length: > 170 mm(6.69in.)
- 2 Mask expansion width: 165 mm(6.49in.). Post-stack width is 95 mm(3.74in.) with three stacks in the middle. Each stack width is not less than 10 mm(0.39in.). The nose clip is located on the upper edge of the mask, and the outside of the mask is darker in color.
- 3 The width of the support part is not more than 10 mm(0.39in.), and the position of the upper nose clip is not more than 16 mm(0.63in.).
- 4 Nose clip: length > 80 mm(3.15in.); width approximately 3 mm(0.12in.)
- 5 Ear loop: 180 mm(7.09in.) in length and 3 mm(0.12in.) in diameter, composed of polyester, spandex and other materials, welded on the inner layer not more than 10 mm(0.39in.) from the edge.

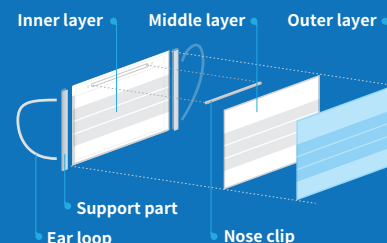

Parameter of BYD Single-use Face Mask

Name	Parameter
Product name	Single-use Face Mask
Material	Polypropylene spunbond nonwoven fabric, polypropylene melt-blown nonwoven, metal wire with plastic covering nose clip, polyester/nylon spandex blend ear loops
Product model	FLAT
Size	175 mm×95 mm / 6.89in.×3.74in.
Expiration date	2 years after production
Packaging specification	10pcs/bag; 50pcs/box; 2000pcs/case
Storage	Store in a room with relative humidity level of no more than 80%, no corrosive gas and good ventilation to avoid high temperature.

WARNING !

- 1 The mask does not eliminate the risk of contracting any disease or infection.
- 2 Failure to properly use and maintain this product could result in illness or even death.
- 3 Only for one-time use, please destroy and dispose according to regulations after use.
- 4 Please use within the validity period.
- 5 DO NOT use if package is damaged.
- 6 For adult use only.
- 7 Use this product immediately after the package is opened. Avoid hand contact within the interior part of the mask. Do not wash.
- 8 Intended for use in environments with moderate fluid exposure.
- 9 This face mask is authorized for distribution under FDA enforcement discretion only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of face masks under section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.
- 10 THIS MASK IS NOT A RESPIRATOR OR A SURGICAL MASK. It is NOT FDA cleared or NIOSH approved and it may or may not meet fluid barrier or filtration efficiency levels. It is recommended for use only by the general public. It is NOT recommended for use in: (1) any surgical setting or where exposure to liquid, bodily or other hazardous fluids, may be expected; (2) a clinical setting where there is an infection risk through inhalation exposure; or (3) the presence of a high intensity heat source or flammable gas. This product does NOT eliminate the risk of contracting any disease, infection or virus.

Time use limitation

If the mask becomes damaged, soiled, or breathing becomes difficult, leave the contaminated area and replace and refit with a new mask.

Important notice

To the extent permitted by law, BYD shall not be liable for any loss or damage including any loss of business, loss of profits, or for any indirect, special, incidental or consequential loss or damage arising from reliance upon any information herein provided by BYD. Nothing in this statement will be deemed to exclude or restrict BYD's liability for death or personal injury arising from its negligence.

All information presented in this document is based on the latest data available at the time of printing. BYD reserves the right to make changes at any time without prior notice.

Approval and Standard

YY/T 0969-2013 Australia TGA ARTG 332299



Distributed by Global Healthcare Product Solutions, LLC

1800 S. Figueroa Street, Los Angeles, CA 90015
1(800)293-2886 www.byd.care Made in China



Scan and visit
BYD Care website



Scan and download
BYD Care app