



SINGLE-USE FACE MASK FOR CHILDREN

Suitable for Ages 6-12

BYD CARE

SOFT & EASY TO BREATHE




Application

Designed for children between the ages of 6 - 12 to cover the mouth, nose and chin, the mask provides a physical barrier. Do not use in a clinical setting where the infection risk level through inhalation exposure is high.



Product Introduction

BYD Single-use face mask for children consists of three layers of nonwoven material:

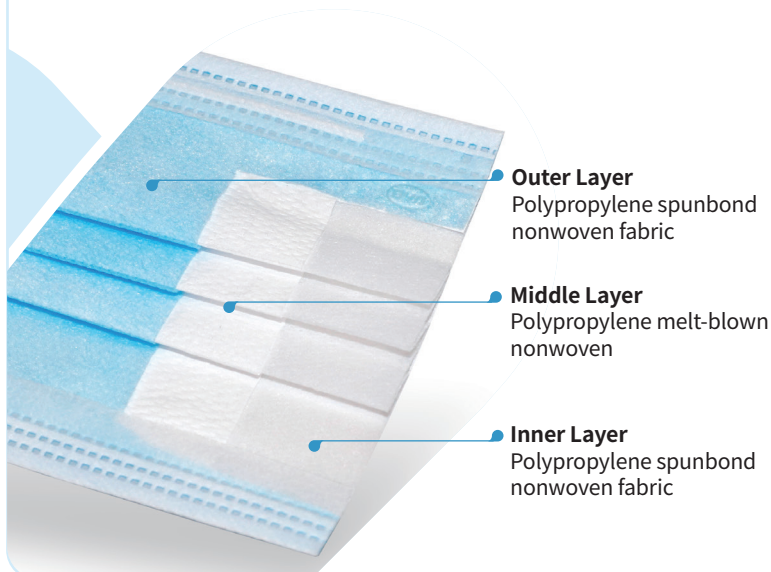

Outer Layer
Polypropylene spunbond nonwoven fabric


Middle Layer
Polypropylene melt-blown nonwoven

Inner Layer
Polypropylene spunbond nonwoven fabric

MATERIALS

Inner layer	Polypropylene spunbond nonwoven fabric
Middle layer	Polypropylene melt-blown nonwoven
Outer layer	Polypropylene spunbond nonwoven fabric
Ear loops	Polyester/nylon spandex blend
Nose clip	Plastic

Not made with natural rubber latex.
Expiration Date: 2 years after production.

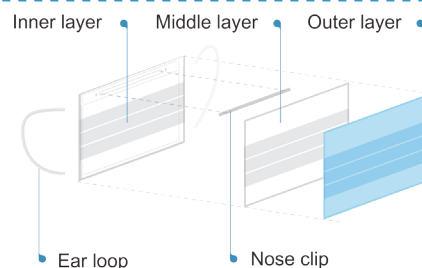

SINGLE-USE FACE MASK FOR CHILDREN



Suitable for Ages 6-12

Instructions for use



1. Wash your hands before touching the mask.



2. Ensure the colored side of the mask faces outwards, the nose clip upwards, and secure the ear loops on each side.



3. Bend the nose clip to match the shape of the nose.



4. Pull the bottom of the mask under your chin to ensure a secure fit.



Do not touch mask once in position. Discard mask if wet or soiled.

Parameter of BYD Single-use face mask for children

Name	Parameter
Product Name	Single-use face mask for children
Material	Polypropylene spunbond nonwoven fabric, polypropylene melt-blown nonwoven, plastic strip nose clip, polyester/nylon spandex blend ear loops
Product model	FE2411
Size	145mm×95mm / 5.71in.×3.74in.
Expiration date	2 years after production
Specifications	10pcs/bag; 50pcs/box; 2000pcs/case
Storage	Store the mask in the original package, away from contaminated areas, dust, sunlight, extreme temperature, moisture and harmful chemicals. Store between -20°C and 30°C (-4°F and 86°F) with relative humidity below 80%.
Colour	Blue/ Pink

WARNING

- 1.The mask does not prevent the risk of contracting any disease or infection.
- 2.Failure to properly use and maintain this product could result in illness or even death.
- 3.Only for one-time use, please destroy and dispose according to regulations after use.
- 4.Please use within the validity period.
- 5.DO NOT use if package is damaged.
- 6.People who are allergic to nonwovens should use with caution.
- 7.Children with dyspnea are not recommended to wear masks. Please follow doctor's advice or wear other respiratory protective equipment.
- 8.Children should wear a mask only under adult supervision and make sure the mask is worn correctly.
- 9.Children should not play or exercise at moderate or intense levels while wearing a mask. If you notice breathing difficulties or skin allergies appear while wearing the mask, please take it off immediately and seek for medical attention if necessary.
- 10.Masks should be stored in a cool and dry place.
- 11.Reusing or washing the mask is not recommended.
- 12.Used masks should not be exchanged with others.
- 13.Please refer to the instructions before use.
- 14.THIS MASK IS NOT A RESPIRATOR OR A SURGICAL MASK. It is NOT FDA cleared or NIOSH approved and it may or may not meet fluid barrier or filtration efficiency levels. It is recommended for use only by the general public. It is NOT recommended for use in: (1) any surgical setting or where exposure to liquid, bodily or other hazardous fluids, may be expected; (2) a clinical setting where there is an infection risk through inhalation exposure; or (3) the presence of a high intensity heat source or flammable gas. This product does NOT eliminate the risk of contracting any disease, infection or virus.

Limitations of Use

If the mask becomes damaged, soiled, or breathing becomes difficult, replace and refit with a new mask.

All information presented in this document is based on the latest data available at the time of printing. BYD reserves the right to make changes at any time without prior notice.

- o The product has not been FDA cleared or approved.
- o The product has been authorized by FDA under an EUA for use as source control by the general public and HCP to help prevent the spread of infection or illness during the COVID-19 pandemic.
- o This product is authorized only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of medical devices, including alternative products used as medical devices, during the COVID-19 outbreak, under section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner.

Standard

T/GDMDMA 0005-2020

Distributed by Global Healthcare Product Solutions, LLC

1800 S. Figueroa Street, Los Angeles, CA 90015
1(800)293-2886 www.byd.care Made in China



Scan and visit
BYD Care website



Scan and download
BYD Care app

Not recommended for more than 4 hours of use.